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**Instructions for Use:**

**Description**

Signigel Electrode Gel is a non-sterile, multipurpose, electrically conductive, water-soluble, non-staining, non-gritty electrode gel. It is bacteriostatic.

**Indications for Use**

Signigel is recommended for all electromedical procedures including ECG, defibrillation, biofeedback and EMG.

**Warnings**

• Do not use on patients with a known history of skin allergies to parabens.

**Precautions**

• For external use only on intact, unbroken skin.  
 • Discard product if package is damaged or evidence of contamination is present.  
 • To avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

**Directions for Use**

Refer to the Instructions For Use provided by the manufacturer of the electromedical device being used for information on how Signigel should be used with their product. Signigel is water soluble, and can be removed using a moistened soft cloth.

**Handling**

No special handling required. Signigel is non-flammable and non-corrosive.

**Storage**

Protect product from direct sunlight.

**Disposal**

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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